visomat[®] handy IV

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Instruction manual (30-55)

Mode d'emploi (56-66)

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Manual de instrucciones (78-87)

CE 0123

Inhaltsangabe

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Contents

A	Description of device	32
B	Brief instructions for use	34
C	Display unit operation	
	1. Setting up the display unit	37
	2. Fitting the cuff	37
	3. Correct posture of device	38
	4. Inflating the cuff	40
	5. Blood pressure measurement	42
	6. Memory recall	43
	7. Memory deletion	43
D	Important information	
	1. Basic rules for blood pressure measurement	44
	2. Other points to note	44
	Circumference of arm	44
	Heart rhythm malfunctions	44
	Frequency of cardiac pacemaker	45
	Pregnancy	45
	Medicines	45

E	What you should know about blood pressure	
	1. Systolic and diastolic blood pressure values	46
	2. Reasons for measuring different values	46
	3. Why you should measure regularly	47
	4. What are standard values?	47
F	Maintenance	
	1. Battery exchange	48
	2. Cleaning and disinfecting	49
	3. After-sales service	49
G	Appendix	
	Malfunctions	49
	Technical specifications/accessories/replacement parts	51
	CE-mark	53
	Metrological inspection	54
	Manufacturer's guarantee	54

- 1 Display for systole, diastole, pulse and control readings
- 2 Start/Stop button
- 3 Memory button (memory recall button)
- 4 Battery compartment
- 5 Cuff

visomat® handy IV:
Digital display unit with electronic pump for measurement of blood pressure on the wrist.

Check display symbols

No.  Memory data (No. of measurement)

A 13 Average No. of stored data

Pulse rate mark

Battery exchange

SYS

Systole

DIA

Diastole

PUL

Pulse

Inflation / Deflation

Please read detailed instructions prior to first measurement! (Page 37)

1. Insert batteries

(2 x 1.5 V alkaline-manganese batteries)

2. Fitting the cuff

Bare your left wrist. With the palm of your left hand up, put the cuff on your wrist so that the main body is on the same side as your palm. Adjust the cuff, with your palm up, until its edge is positioned 5 to 10 mm (1/2 to 1/1 in.) from the lowest part of your palm (Fig.1).

Fasten the cuff around your wrist so that there is no space between the cuff and

your wrist. The cuff should fit snugly. Stretch your left arm out and rest it on the table with your palm facing upwards (Fig. 2). Keep your arm still, and do not talk. Press the surface of the cuff to make sure that it is attached securely.

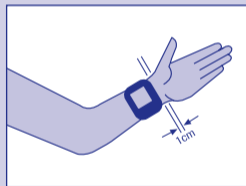
Fig.1

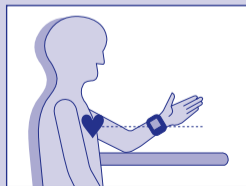
Fig.2

3. Pumping up the cuff

Press the "Start/Stop" button.

The machine will carry out a function test, and all display elements will be visible. The cuff will be pumped up automatically to about 190 mmHg. If the pressure in the cuff is still too low, the machine will continue to pump it up in steps of about 30 mmHg at a time.

4. Blood pressure measurement

After inflation, the air escapes automatically, the falling pressure in the cuff will be shown automatically, and the "♥-symbol" will flash and beep.

The automatic release of the air from the cuff will show the end of the measuring cycle. The systolic and diastolic blood pressure readings will appear in the display alternating in time with your pulse.

5. Switching the machine off

Press the Start/Stop button. The readings will be stored in the memory automatically.


1. Setting up the display unit

Load enclosed batteries, if this has not already been done (see “Battery exchange”).

2. Fitting the cuff

Bare your left wrist.

Left-handed users can fit the cuff to their right arm (observing the general sense of instructions). With the palm of your left hand up, put the cuff on your wrist so that the main body is on the same side as your palm. Adjust the cuff, with your palm up, until its edge is positioned 5 to 10 mm (1/4 to 1/2 in.) from the lowest part of your palm. Fasten the cuff around your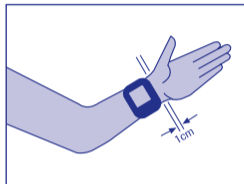


wrist so that there is no space between the cuff and your wrist. The cuff should fit snugly. The cuff sensor underside should now lie directly on your pulse. Press the surface of the cuff to make sure that it is attached securely. With its improved shape the cuff fits tightly to prevent measurement failures.


3. Correct posture of device

3 postures for measurement are possible:

1. Stretch your left arm out and rest it on the table with your palm facing upwards. Bend the arm to the same height as your heart.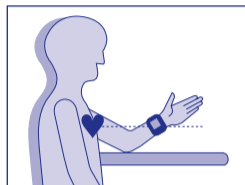

2. If you once do not measure at a table, you can measure in a posture as follows: After pressing the Start/Stop button lay our arm on your body so that the cuff is at the same height as your heart. In order to relieve the left arm you can rest it on your right arm. Do not move until measurement has finished.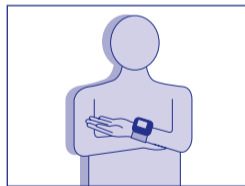


3. Making a Measurement in Lying

Lie on your back. Adjust the cuff to the same height as your heart (e.g. with a pillow).

Breathe deeply 5 or 6 times to relax before measurement.

Measured data may differ very slightly depending on the posture during measurement.

Keep your arm still, and do not talk.

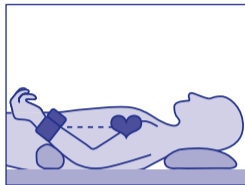Correct measurements can only be achieved when the cuff is adjusted at the same height as your heart. If the cuff is lower (higher) in position than the heart, the measured reading tends to become larger (smaller).

4. Inflating the cuff

Press the Start/Stop button. The machine will carry out a function test, and all display elements will be visible. First the display shows an arrow down, then "0" and the cuff will be pumped up automatically to about 190 mmHg. If the pressure in the cuff is too low, the machine will continue to pump it up in steps of 30 mmHg at a time.

Inflating the cuff and measuring can be stopped at any time by pressing the Start/Stop button again.

To manually control pressurization, hold down the Start/Stop button immediately after the start of inflation and release it

when the pressure reaches the level 30 mmHg higher than the expected maximum systolic pressure. If the pressure value has exceeded 190 mmHg, the inflation will stop when the key is released. Pressure can be increased to approximately 300 mmHg.

Attention:
Do not inflate the cuff if it is not fitted to your arm.

5. Blood pressure measurement

Once the necessary pressure in the cuff is reached, the pump stops and the air escapes automatically. The falling cuff pressure appears in the display.

After a short time, the symbol "♥" flashes in time with your pulse. At the end of the measurement the remaining air escapes automatically from the cuff. The display unit then shows the measured value of the blood pressure alternating in time with the pulse speed. When a new measurement is started, or the device is switched off, the values of systolic and diastolic blood pressure are automatically memorized.

Enter the memorised value in your blood pressure chart. This will provide you with an overview of your blood pressure situation for a longer period, which you can discuss with your doctor if necessary.

Please wait 3 minutes before taking a new measurement.

6. Memory recall

The device is equipped with a memory which is able to show the last 30 values of the blood pressure and pulse measurements as well as an average value.

When the memory button is pressed, the average value appears first (as far as 2 values are stored). When the memory button is pressed again the last measured value (no. 1) appears. After pressing the button a third time the second latest memory (no. 2) appears, and so on.

With this procedure all 30 memories can be recalled. Finally, another pressing of the memory button recalls the average of all measured values.

After the 30th measurement, the oldest measurement will be deleted and the latest measured value will be memorised.

7. Memory deletion

Single memorized values can be deleted when the memory button is pressed several times until the wanted value appears. When the button is pressed 4 seconds the value will be deleted. To delete the complete storage the memory button must be pressed until the display is blank (7-8 sec).

When the data on the display flashes the deletion was not completed and has to be repeated.

1. Basic rules for blood pressure measurement

- The measurement must be carried out whilst you are relaxed. Walking or going upstairs increases the blood pressure. Please wait at least 5 minutes until your pulse rate has returned to normal.
- Check the blood pressure twice a day – in the morning after getting up and in the evening after work when you are relaxed.
- To carry out the measurement, sit down

at your dining table. This ensures that the cuff is kept at heart level.

- Perform the measurement on your bare arm to reach exact values.
- Do not move the arm while wearing the cuff.
- Please wait at least 3 minutes between successive measurements in order to avoid measurement errors.

2. Other points to note

- Circumference of arm
The cuff is made to fit an arm circumference of 12.5–21.5 cm.

Important information

- **Heart rhythm malfunctions**
False results can be obtained if you are suffering from heart rhythm malfunctions or arrhythmia. Pay attention to regular beeps during measurement.
- **Frequency of cardiac pacemaker**
The pulse frequency display is not suitable for checking the frequency of cardiac pacemakers.
- **Pregnancy**
During pregnancy blood pressure measurement should not be performed without prior consultation of your doctor since the pregnancy might affect the blood pressure levels.

- **Medicines**
If you are taking blood-thinning/anti-coagulating drugs (clot prevention) you should only make a measurement after consulting your doctor.

Measuring your own blood pressure is not the same as treatment! Do not change any drug dosage determined by your doctor.

1. Systolic and diastolic blood pressure values

The cardio vascular system has the important function of supplying all organs and tissue in the body with sufficient blood and of eliminating the products of metabolism. The heart contracts and relaxes in a regular rhythm around 60–80 times per minute. The pressure of the blood applied to the artery walls during contraction of the heart is described as **systole**. The pressure in the subsequent relaxed phase, when the heart fills up with blood again, is described as **diastole**. You determine both values in your daily measurements.

2. Reasons for measuring different values

Our blood pressure reacts like a sensitive measurement instrument to external and internal influences. Even slight alterations can have an effect. This explains why measurements taken at your doctor's or chemist's are often higher than those taken at home. But also such factors as weather changes, change of climate, physical and mental stress can affect the measurements.

3. Why you should measure regularly?

Even the time of day has an influence on your blood pressure. During the day the values are generally higher than during the period of relaxation at night. One-off and irregular measurements therefore say little about your daily blood pressure. A reliable assessment is possible only when measurements are taken regularly. Speak to your doctor about the results.

High blood pressure	Borderline	Normal pressure
systolic pressure $\geq 140^*$ and/or**	systolic pressure 120–139	systolic pressure ≤ 119
diastolic pressure ≥ 90	diastolic pressure 80–89	diastolic pressure ≤ 79

2003

* mmHg = Millimetres of hydragyrum (mercury)

** It suffices for one of the values to be raised

4. What are standard values?

The World Health Organization (WHO) has drawn up the following overview for assessing blood pressure values:

1. Battery exchange

- **Open battery compartment**

Open the cover on the side of the device by applying light pressure and pushing in the direction of the arrow.

- **Insert batteries**

Remove the old batteries from the display unit and insert the new ones. Make sure that the polarity is correct (marks inside of the battery compartment).

- **Close battery compartment**

Close the battery compartment by replacing the cover.

- **Please note:**

Replace the batteries when:

- The battery replacement indication appears on the display.
- Nothing is displayed even if the power button is depressed.

Always exchange both batteries. Use only high quality batteries as indicated in the technical specifications.

If the display unit is not going to be used for a longer period, remove the batteries to prevent leakage.

- **Dispose of batteries and technical devices at a recycling point in your area.**

2. Cleaning and disinfecting

Clean the display unit only by using a soft, slightly damp cloth and some detergent. Stains on the cuff can be removed carefully in the same way.

Please note:

Do not wash or dry-clean the cuff.

3. After-sales service

Any repairs to the display unit may only be carried out by the manufacturer/ supplier or by a service centre authorized by UEBE.

Please contact
Uebe Medical GmbH
Zum Ottersberg 9
97877 Wertheim/Germany
Phone: +49 / 93 42 / 92 40 40
Fax: +49 / 93 42 / 92 40 80
E-Mail: info@uebe.com
Internet: www.uebe.de

Malfunctions

In rare cases, you may receive no results or false results of measurements. Your display unit shows you a code which will help you to rectify malfunctions (see list on page 50)

Malfunctions shown	Reasons	Corrective Action
Symbol ERR 1 is displayed – Measurement error	Movement of the arm or body during the measurement. Do not talk during the measurement	Repeat the measurement. Don't move the arm and don't talk. In case of recurrence please contact our after-sales service
ERR 2 – inflation error	Pump problem	Repeat measurement. In case of recurrence please contact our after-sales service
ERR 3 – deflation error	Deflation rate too high or too low, valve problem	In case of recurrence please contact our after-sales service
ERR 330	Air pressure exceeds 330 mmHg Battery problem	Repeat measurement Check batteries
No measuring or check displays on the screen	No battery inserted – Battery incorrectly inserted – Flat battery	Please check battery and replace if necessary
Battery symbol 	Battery is flat	Please check battery and replace if necessary
Faulty measurement values	Moving your arm or body during the measurement.	Repeat the measurement. Don't move the arm and don't talk.
	Too low or high position of wrist	Repeat measurement with correct posture
	Cuff does not fit Cuff is not put on correctly	Check wrist circumference Repeat measurement
	Heart rhythm malfunctions during measurement	Repeat measurement
Cuff is not pumped	Battery is flat	Please check battery and replace if necessary.

Technical specifications/ accessories/replacement parts

- **Display unit type:**
Digital display unit with electric pump for measurement of blood pressure on wrist.
- **Measuring system:**
Oscillometric determination of systole, diastole and pulse value
- **Reference procedure of clinical investigation:**
Auscultatory measurement
- **Display:**
LCD-display for 7-digit measuring parameters and check display
- **Memory:**
30 measurements (saved automatically) and the average value
- **Measuring range:**
Blood pressure measurement:
0–300 mmHg
Pulse measurement:
40–160 pulse/min.
- **Error range:**
 - Blood pressure measurement:
correspond to EN 1060 Part 3
 - Pressure measurement: ± 3 mmHg
 - Pulse measurement: $\pm 5\%$
- **Type of cuff:**
Shell-shaped cuff

- **Deflation valve:**
Electronically regulated linear valve
 - **Deflation rate:**
average 5,5 mmHg
 - **Inflating pressure:**
Approx. 190 mmHg
 - **Battery:**
 - Battery type:
2 x AAA alkaline-manganate block cell (LR 03) or lithium (FR 03)
 - Durability:
More than 600 measurements in 2 years
 - Battery check:
Display "  " means exchange battery
 - Automatic power-off:
About 3 minutes after end of measurement
 - **Conditions of use:**
Temperature between 10–40°C
Humidity 15–90%
 - **Accessory**
Shell-shaped cuff "Type H4"
(12,5 – 21,5 cm)
Art.No. 2104001
- Important:***
Use the display unit only with original cuff Type H4.

CE-mark

This blood pressure display unit corresponds to the European regulation of EC directive 93/42/EEC on medical devices and bears the CE-mark "CE 0123".

Reference procedure of clinical investigation: auscultatory measurement.

Protection grade against electric shock: Type BF

note instruction manual

Applied standards:

- EN 1060 Part 1: 1995
Non-invasive sphygmomanometers
Part 1: General requirements
- EN 1060 Part 3: 1997
Non-invasive sphygmomanometers
Part 3: Supplementary requirements for electromechanical blood pressure measuring systems
- prEN 1060-4: 2002
Non-invasive sphygmomanometers (clinically validated)
- EN 60601-1:1990
+A1:1993
+A2:1995
 1. Medical electrical devices
 2. General commitment for safety
- EN 60601-1-2:2001
Medical electrical devices
Electromagnetic tolerance

Producer UEBE Medical GmbH
Zum Ottersberg 9
97877 Wertheim/Germany

Metrological inspection

(for professional users)

A metrological inspection corresponding to the "Regulations for the Retail of Medical Products" should be carried out every two years either by Uebe Medical GmbH, a public authority responsible for weights and measures or by an authorised service centre.

On request, the responsible authority or maintenance service will be sent a copy of the "Instructions for metrological inspections" by the manufacturer.

Manufacturer's guarantee

The equipment was manufactured and tested with great care. However, in the unlikely event of a defect being detected after delivery, we are prepared to give a guarantee in accordance with the following conditions:

1. During the guarantee period of 3 years from the date of purchase, we reserve the right to either repair any such defect at our expense or supply a perfect replacement unit (in any case the defective equipment must be returned to our factory).
2. Excluded from the guarantee are parts subject to normal wear and tear as well as damage caused by non-compliance with the operating instructions, improper

handling (for example: unsuitable supply mains, breakages, leaking batteries) and/or dismantling of the unit by the purchaser. Furthermore, no claims for damages against us can be justified through the guarantee.

3. Claims under the guarantee can only be made during the guarantee period by presentation of the purchase receipt. In the case of claim under the guarantee, the equipment together with the purchase receipt and a written justification of the claim are to be sent to UEBE Medical GmbH, Service-Center, Zum Schlag 18, 97877 Wertheim, Germany.

4. In the case of defectiveness of the goods, the contractual rights of the purchaser to claim against the supplier/salesman in accordance with § 437 BGB (The German Civil Code) are not limited by the guarantee.

Please note:

In a warranty case, please return the entire unit together with the sales slip.

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© Copyright UEBE Medical GmbH