B.Well

caring for everyone

EN	Instructions for Use	3
BG	Инструкции за използване	9
ET	Kasutusjuhend	15
LT	Naudojimo instrukcijos	21
LV	Lietošanas instrukcijas	27
NO	Bruksanvisninger	33
PL	Instrukcja użytkowania	39
RO	Instrucțiuni de utilizare	45
SR	Uputstva za upotrebu	51
SV	Användningsanvisningar	57
AZ	İstifadəyə dair təlimatlar	63
RU	Инструкция по использованию	69
UK	Інструкція з використання	75

MED-50 / MED-80

Digital upper arm blood pressure monitor

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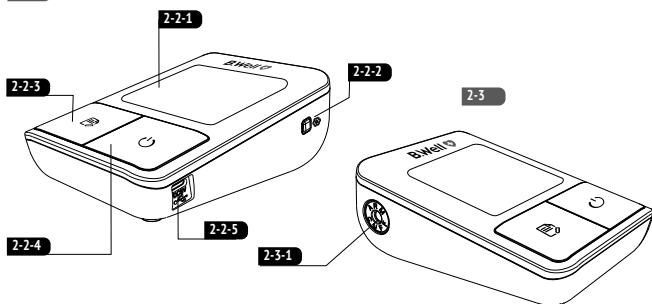
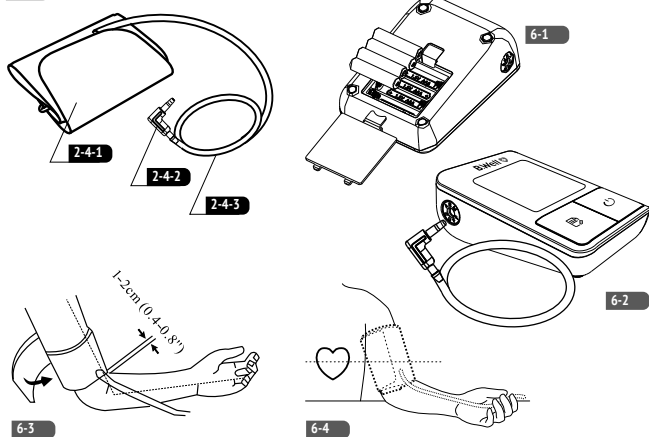
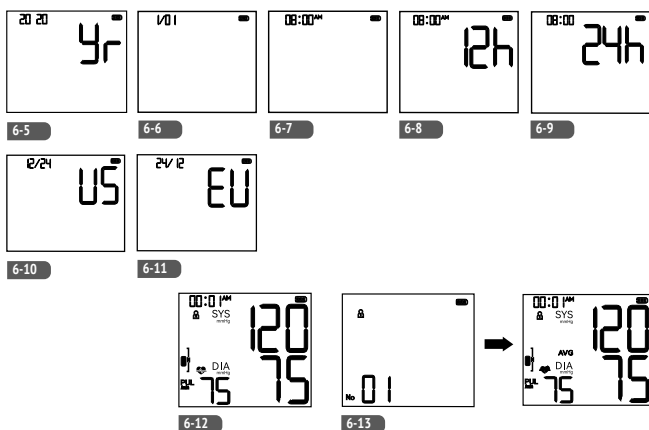
Instructions for Use

1. INTRODUCTION

Thank you for purchasing the B.Well upper arm blood pressure monitor MED-50 / MED-80.

PRODUCT DESCRIPTION, INTENDED USE AND APPLICATION AREA

This device is intended for non-invasive measuring adults and adolescents over 12 years of age individual's systolic, diastolic blood pressure and heart rate using the oscillometric method. The device is not intended for use on infants and children. The device is designed for home or clinical use. Measurement position is on upper-arm only.

Intended patient population: >12 years of age, i.e. adolescents age 12 through 18 years of age, adults >18 years of age.

⚠ In case of any serious incident with the device, immediately contact the manufacturer and national competent authorities.

IMPORTANT ADVANTAGES OF MED-50 / MED-80

- Accuracy control indicators: cuff loose, arm shake.
- Comfortable anatomic cuff with uni cuff size 22-42 cm fits almost every arm.
- Correct monitoring as recommended by the doctor – calculation of the average of the last 3 measurements.
- Memory of 100 values with date and time for 1 user.
- Color scale of pressure level.
- Pulse arrhythmia indicator.
- Ergonomic design and solid screen protection.
- USB-C cord.

⚠ PRE-APPLICATION INFORMATION

Please read this manual thoroughly before using the unit. Please retain this manual for future reference. For specific information about your blood pressure, please CONSULT YOUR DOCTOR. To avoid risk and damage follow all warning precautions. Operate unit only as intended.

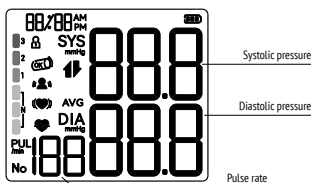
The use of third-party components (cuffs, air tubes and connectors) may negatively affect the operation of the device, lead to failure of the device algorithm and cause harm to the user. Batteries, USB cables and adapters, if required when using this device, must comply with the requirements specified in the instructions for use. Use of unsuitable power supply elements (batteries, USB cables) may cause electromagnetic interference and impair operation of the device.

MED-50 / MED-80

2. CONTENTS AND DISPLAY INDICATORS

Watch screen, device and cuff pics (see pictures on pages 2 and 3).

LCD SCREEN 2-1

- 1/2 Cuff inflation / deflation
- AVG Last 3 tests average
- Group
- Cuff loose indicator
- Arm shake indicator
- Pulse arrhythmia indicator
- Heart rate indicator
- Electric quality indicator
- WHO blood pressure classification indicator

DEVICE 2-2 2-3

- 2-2-1 LCD
- 2-2-2 SET button
- 2-2-3 MEMORY button

EN

Instructions for Use

2-2-4 START/STOP button**2-2-5** USB-C jack**2-3-1** Cuff socket**CUFF 2-4****2-4-1** Arm cuff**2-4-2** Air plug**2-4-3** Air tube**3. CONTRAINDICATION**

- ⚠ Product is not intended for infants or individuals who cannot express their intentions.

4. ABNORMAL REACTION

Not detected.

5. PRECAUTIONS AND WARNINGS**5.1. Precautions before and during use**

- ⚠ Before use, please read this instruction manual carefully.
- ⚠ Do not use this unit in a moving vehicle. This may result in erroneous measurement.
- ⚠ Motion, trembling, shivering may affect the measurement reading.
- ⚠ This device is not used for children and pets.

5.2. Medical precautions

- ⚠ Do not confuse self-monitoring with self-diagnosis. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
- ⚠ Individuals with serious circulation problems may experience discomfort. Consult your physician prior to use.
- ⚠ For persons with irregular or unstable circulation resulting from diabetes, liver disease, arteriosclerosis or other medical conditions, there may be variations in blood pressure values measured at the wrist versus at the upper arm. Monitoring the trends in your blood pressure taken at either the arm or the wrist is nevertheless useful and important.
- ⚠ People suffering from vascular constriction, liver disorders or diabetes, people with cardiac pacemakers or a weak pulse, and women who are pregnant should consult their physician before measuring their blood pressure themselves. Different values may be obtained due to their condition.
- ⚠ People suffering from arrhythmias such as atrial or ventricular premature beats or atrial fibrillation only use this blood pressure monitor in consultation with your doctor. In certain cases oscillometric measurement method can produce incorrect readings.
- ⚠ The cuff should not be placed on the arm on the side of a mastectomy. In the case of a double mastectomy use the side of the least dominant arm.

5.3. General precautions

- ⚠ The time required for the device to warm from the minimum storage temperature (-25 °C) between use until the device is ready for use at ambient temperature (20 °C): about 2 hours.
- ⚠ The time required for the device to cool from the maximum storage temperature (55 °C) between use until the device is ready for use at ambient temperature (20 °C): about 2 hours.

5.4. Warning/Safety notice

- ⚠ Contact your physician if test results regularly indicate abnormal readings. Do not attempt to self treat these symptoms without consulting your physician first.
- ⚠ Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
- ⚠ If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.
- ⚠ Product is designed for its intended use only. Do not misuse it in any way.

MED-50 / MED-80

- ⚠ Too frequent measurements can cause injury to the patient due to blood flow interference.
- ⚠ DO NOT attach the cuff to a limb being used for IV infusions or any other intravascular access, therapy or an arterio-venous (A-V) shunt. The cuff inflation can temporarily block blood flow, potentially causing harm to the patient.
- ⚠ The cuff should not be applied over a wound as this can cause further injury.
- ⚠ Pressurization of the cuff can temporarily cause loss of function of simultaneously used monitoring equipment on the same limb.
- ⚠ A compressed or kinked connection hose may cause continuous cuff pressure resulting in blood flow interference and potentially harmful injury to the patient.
- ⚠ Check that operation of the unit does not result in prolonged impairment of the circulation of the patient. Prolonged over-inflation of the bladder may cause ecchymosis of your arm.
- ⚠ Use only the approved arm cuff for this unit. Use of other arm cuffs may result in incorrect measurement results.
- ⚠ The system might produce incorrect readings if stored or used outside the manufacturer's specified temperature and humidity range.
- ⚠ Do not use the device near strong electrical or electromagnetic fields generated by cell phones or other devices, they may cause incorrect readings and interference or become interference source to the device. Do not use the device during patient transport outside healthcare facility for interference source existing as well.
- ⚠ Do not disassemble the unit or arm cuff. Do not attempt to repair.
- ⚠ No service and maintenance while it is in use and maintenance only be performed by service personnel.
- ⚠ Do not mix new and old batteries simultaneously. Do not mix battery types. Do not insert the batteries with their polarities incorrectly aligned. Long-life alkaline batteries are recommended.
- ⚠ Remove batteries from device when not in operation for more than 3 months. Dispose batteries properly; observe local laws and regulations.
- ⚠ Replace batteries when Low battery indicator  appears on screen. Replace all 4 batteries at the same time.
- ⚠ Only use a recommended AC adapter double-insulated complying with EN 60601-1 and EN 60601-1-2. An unauthorized adapter may cause fire and electric shock.
- ⚠ It is recommended to use B-Well adapter AD-51/AD-81/AD-75.
- ⚠ Please place the blood pressure monitor out of the reach of children.
- ⚠ Advising operator that instruction manual / booklet must be consulted.
- ⚠ This device is not used for children and pets.
- ⚠ If pressure in arm cuff becomes too extreme while testing, press the START/STOP button to turn power off. The cuff pressure will rapidly dissipate once the unit is off.
- ⚠ Do not place the blood pressure monitor and cuff at will. It will cause asphyxiation if the child swallows or twists the cuff around his neck.

6. SETUP AND OPERATING PROCEDURES**6.1. Battery loading and replacement**

After unpacking the device install the batteries.

The battery compartment is located on the underside of the case.

Battery Installation:

1. Slide battery cover off.
2. Install 4 new AAA alkaline batteries according to polarity.
3. Close battery cover.

6-1

- ⚠ If the batteries are low, the corresponding symbol  will appear on the screen. The appliance will then not switch on. Replace all batteries with new ones.
- ⚠ Do not leave discharged batteries inside the device. Also remove the batteries if the device will not be used for a month or more to avoid possible damage to the device in case of battery leakage.
- ⚠ Avoid contact of electrolyte with eyes. If electrolyte gets into eyes, rinse immediately with plenty of clean water and consult a doctor.

6.2. Connecting components

A. Connecting the AC adapter/cable

- The device has a USB-C connector for connecting the AC power adapter/cord, which is located at the right side panel of the device.
- Plug the USB-C adapter/cord into a 100–240 V, 50–60 Hz AC mains supply.
 - Connect the plug of the AC adapter/cord to the appropriate connector.
The battery power of the unit is automatically switched off.

① Note:

If the AC adapter or micro USB-C cord is not included, it can be purchased separately. Use only the AC adapter/cord recommended by B. Well. If any other adapter is used, the service centre will not be liable for warranty service.

- ⚠ Do not plug or unplug the power cord into the electrical outlet with wet hands.
- ⚠ Do not overload power outlets. Plug the device into the appropriate voltage outlet.
- ⚠ If the AC adapter is abnormal, please change the adapter.
- ⚠ Do not use any other type of AC adapter as it may harm the monitor.
- ⚠ Please use the adapter with characteristics:
INPUT: 100–240V-50/60Hz, 0.2A; OUTPUT: 5.0V, 1.0A, 5.0W

B. Connecting the arm cuff to the monitor

Firmly insert air tube plug into opening located on the right side of monitor unit. Make certain that the Plug is completely inserted in order to prevent air leakage during use.
With sticky nylon section facing outward, insert end of cuff underneath metal ring of cuff.

6-2

- ⚠ Avoid compression or restriction of the connection tubing during measurement, which may cause inflation error, or harmful injury due to continuous cuff pressure.

6.3. Applying the cuff

- Thread the end of the cuff through the metal ring so that the entry point of the air tube into the cuff is on the outside.
- Slide your arm through the cuff so that the air tube is positioned along your arm towards your palm.
- Fasten cuff about 1–2cm (0.4–0.8") above the elbow joint.
- Tighten the cuff firmly but not too tightly, taking into account the taper of the arm, by pulling the free end.
- The cuff must fit tightly around the arm, otherwise the measurement result will be incorrect. The cuff must be at the level of the heart. It is not recommended to wear the cuff over clothing.
- Place your hand with the palm of your hand facing upwards. Make sure the pump tube is not twisted, compressed or restricted.

6-3

6-4

- ⚠ The Artery Mark on the cuff indicates the location of the artery on the left arm, as this is the most common arm for measuring blood pressure.

6.4. System settings

With power off, press SET button to activate system settings.

A. Time/Date setting

Press the  button to set the Time/Date mode. Set the year first by adjusting the button .

Press the  button again to confirm current month. Continue setting the date, hour and minute in the same way.

Every time the  button is pressed, it will lock in your selection and continue in succession (month, day, hour, minute, 12/24 hours).

6-5

6-11

B. Saved settings

While in any setting mode, press the  button to turn the unit off. All information will be saved.

① Note:

If unit is left on and not in use for 3 minutes, it will automatically save all information and shut off.

6.5. Carrying out a measurement

Before measuring

- Avoid eating, exercising, and bathing for 30 minutes prior to testing.
- Sit in a calm environment for at least 5 minutes prior to testing.
- Do not stand while testing. Sit in a relaxed position while keeping your arm level with your heart.
- Avoid speaking or moving body parts while testing.
- While testing, avoid strong electromagnetic interference such as microwave ovens and cell phones.
- Wait for 3 minutes or longer before re-testing.
- Try to measure your blood pressure at the same time each day for consistency.
- Test comparisons should only be made when monitor is used on the same arm, in the same position, and at the same time of day.
- This blood pressure monitor is not recommended for people with severe arrhythmia.
- Do not use this blood pressure monitor if the device is damaged.

Measurement in sitting position

- Sit down and place your feet flat on the floor, don't cross your legs or bend them under you. Lean your back straight against the back of the chair.
- Place palm upside in front of you on a flat surface such as table.
- The middle of the cuff should be at the level of the heart.
- ⚠ To avoid errors in the measurement, it is important to remain immobile during the measurement and in silence.

Measuring in the prone position

- Lie on your back.
- Place your left arm straight along your side with your palm upside.
- The cuff should be placed at the same level as your heart.

Common sources of errors or incorrect pressure values

- Movement during measurement.
- Incorrect positioning of the cuff in relation to the heart.
- The cuff does not fit you in size.
- Loosely tightened cuff.

Power on

Press and hold the  button to turn the unit on. The LCD screen will appear for one second as unit performs a quick diagnosis. A voice tone will indicate when unit is ready for testing.

① Note:

Unit will not function if residual air from previous testing is present in cuff. The LCD will flash  until pressure is stabilized.

Testing

After cuff inflation, air will slowly rise as indicated by the corresponding cuff pressure value. A flashing  will appear simultaneously on screen signaling heart beat detection.

① Note:

Keep relaxed during testing. Avoid speaking or moving body parts.

Result display

The screen will display measurements for systolic and diastolic blood pressure. An indicator representing the current measurement will appear next to the corresponding WHO Classification.

6-12

Average value of last 3 results setting

With power off, press the  button to activate screen display. After the unit performs a self-diagnosis, the screen will display the average test results from the last 3 readings. The AVG symbol will appear along with the corresponding WHO blood pressure indicator.

The Memory check mode can be accessed by pressing .

Pulse arrhythmia indicator

When the monitor detects an irregular rhythm during the measurements, the Pulse arrhythmia indicator  will appear on the display with the measurement values. Irregular heartbeat rhythm is defined as rhythm that is either 25% slower or faster than the average rhythm detected while measuring systolic blood pressure and diastolic blood pressure.

Note:

Consult your physician if the Pulse arrhythmia indicator frequently appears with your test results.

TAKING CARE OF MEASUREMENT ACCURACY**Arm shake indicator**

If there is arm movement during the measurement, the  icon may flash. It indicates that the measurement results may be inaccurate, and the situation will be recorded at the end of the measurement as a reminder.

Cuff loose indicator

When starting the measurement,  will be displayed when the cuff is properly wound. When the cuff is too loose,  will be displayed. At this time, please wear the cuff correctly and start measuring again.

Power off

The  button can be pressed to turn off the unit in any mode. The unit can turn off the power itself after about 3 minutes of no operation in any mode.

Safety precaution: If pressure in arm cuff becomes too extreme while testing, press the  button to turn power off. The cuff pressure will rapidly dissipate once the unit is off.

Saved settings

While in any setting mode, press  button to turn the unit off. All information will be saved.

Memory check

This device stores up to 100 measurements in memory with date and time. You may check past test results by using the  button. Press the button to scroll through all test results stored in memory. The most recent test result and oldest test result in memory can be viewed by pressing twice the  button. Upon activating test results, you can press the  button to scroll through all test results stored in memory.

6-13

- △ Numbering of measurement results in the device memory:
1 – the very first, 100 – the last.

Memory deletion

Memory may be deleted while in Memory check mode. Press and hold the  button for approximately 3 seconds to delete and then transfer into testing mode. Press the  button to turn the unit off.

Note:

Memory cannot be recovered once it has been deleted.

Low battery indicator

There will be 4 short warning beeps when battery life is depleting and unable to inflate cuff for testing. The  appears simultaneously for approximately 5 seconds prior to shutting off. Replace batteries at this time. No memory loss will occur throughout this process.

6.6. Blood Pressure levels assessing:**WHO blood pressure classification indicator**

The following guideline for assessing high blood pressure (without regard to age or gender) has been established by WHO. Please note that other factors (e.g. diabetes, obesity, smoking, etc.) need to be taken into consideration.

Range	SBP mmHg	DBP mmHg	Color indicator	Measures
Optimal	< 120	< 80	Green	Self-check
Normal	120–129	80–84	Green	Self-check
High normal	130–139	85–89	Green	Consult your doctor
Grade 1: mild hypertension	140–159	90–99	Yellow	Consult your doctor

MED-50 / MED-80

Grade 2: moderate hypertension	160–179	100–109	Orange	Consult your doctor immediately
Grade 3: severe hypertension	≥ 180	≥ 110	Red	Urgently seek medical advice
WHO / ESH Definition and Classification of Blood Pressure Levels				
SBP – systolic blood pressure, DBP – diastolic blood pressure				

- △ The BP level indication is a reference and does not imply independent emergency measures.
- △ Consult with your physician for accurate assessment, and never change your treatment by yourself.
- The colored bars to the left of the display show the range within which the indicated blood pressure value is located.
- Depending on the height of the indicator, the readout value is either within the normal (green), borderline (yellow and orange) or hazardous (red) range.
- If the systole and diastole values fall into two different categories (e.g., systole in the High Normal category and diastole in the Normal category), the display always shows the higher category.

7. TROUBLESHOOTING**Troubleshooting (1)**

PROBLEM	POSSIBLE CAUSE	SOLUTION
Abnormal sphygmomanometer	The armband is tied too tight or too loose. Or the arm strap is tied incorrectly.	Roll the armband correctly
	Moving the arm during measurement of electronic sphygmomanometer	Stay quiet, keep your arm steady, and do not move the monitor
	Speaking, being nervous or emotional during measurement	Instead of talking, take deep breaths to calm your mood and relax your body
	Incorrect measurement posture	Adjust posture
	There is interference in charging process or improper operation in measuring process	See operation Instructions

- If errors occur during measurement, check the following points and take appropriate action:

Troubleshooting (2)

LCD display symbols	POSSIBLE CAUSE	SOLUTION
Er1	Can't detect high and low pressure	Please fasten the cuff before measuring
Er2	Cuff too loose or loose	Please fasten the cuff before measuring
Er3	Arm or body movements or other behaviors may cause abnormal data during the measurement of blood pressure	Please hold the arm or body still. Please keep quiet and measure again
Er4	The pressure exceeds 300 mmHg	Please fasten the cuff before measuring
Er5	The pressure exceeds 15 mmHg for 3 minutes	Check whether the cuff is knotted. If the problem persists, contact the manufacturer
Er6	Blood pressure measurements were out of range	Please tighten and measure and measure again. If you cannot solve the problem, please contact the manufacturer

Er0	Four sets of results were measured that did not meet requirements of avg last 5	Please tighten and measure and measure again. If you cannot solve the problem, please contact the manufacturer
	Battery dead	Replace the battery or connect the power adapter (if any).

① **Note:**

If you cannot solve the abnormal situation by yourself, you can consult the manufacturer. It is forbidden to disassemble and repair without permission. If necessary, professional maintenance personnel can ask the manufacturer for the list of components and circuit schematic diagram.

8. MAINTENANCE

8.1. Maintenance (general)

- △ Avoid dropping, slamming, or throwing the unit.
- △ Avoid high temperature and direct sunlight. Do not immerse the monitor in water as this will result in damage to the monitor.
- △ Do not attempt to disassemble this monitor.
- △ No modification or repairing of this equipment is allowed for the user during exploitation period. No service and maintenance while it is in use and maintenance only be performed by service personnel.

8.2. Maintenance (operation)

- Remove batteries when not in operation for an extended period of time.
- Monitor can maintain the safety and performance characteristics for a minimum of 10,000 measurements or three years, and the cuff integrity is maintained after 1,000 open-close cycles of the closure.
- It is recommended the performance should be checked every 2 years of regular use or after dropping/damaging. Please contact an authorized service center.

8.3. Maintenance (cleaning)

- When cleaning the unit, use a soft fabric and lightly wipe with mild detergent. Use a damp cloth to remove dirt and excess detergent. Do not use solvents.
- △ **Cuff Cleaning:** Do not soak cuff in water! Apply a small amount of rubbing alcohol to a soft cloth to clean cuffs surface. Use a damp cloth (water-based) to wipe clean. Allow cuff to dry naturally at room temperature. The cuff must be cleaned and disinfected before use between different users.
- The cuff cover can be hand washed at 30°C. Remove the bladder beforehand through the special technological hole in the cuff cover.
- △ The bladder must not be washed!
- △ The cuff cover must not be ironed or dried in the tumble dryer!
- △ The technological **hole must not be sewn up!**

The device, batteries and cuffs must be disposed of in accordance with local regulations at the end of their useful life.

9. SPECIFICATIONS

Product description	Upper arm fully automatic blood pressure monitor
Model	1. MED-50 2. MED-80
Display	70 mm × 59 mm (2.76" × 2.32")
Measuring method	Oscillometric method, automatic air inflation and measurement
Classification	Internal Powered Equipment, Type BF  . Cuff is the Applied Part
Unit dimensions	Approx. 124 mm × 92 mm × 43 mm (4.88" × 3.62" × 1.69")
Cuff circumference	22–42 cm (8 21/32" – 16 17/32")
Unit weight	Approx. 186 g (excluding batteries)
Memory	1 × 100 memories with date and time

MED-50 / MED-80

Power source	Batteries: 4 × 1.5V  AAA DC: 5.0 V  1.0 A Medical AC adapter: INPUT: 100–240V-50/60Hz, 0.2A; OUTPUT: 5.0V, 1.0A, 5.0W
Measurement range	Cuff pressure: 0–299 mmHg Systolic: 60–260 mmHg Diastolic: 40–200 mmHg Pulse Rate: 30–180 beats/minute
Accuracy	Pressure: ±5 mmHg Pulse Rate: ±5%
Environmental temperature for operation	10°C–40°C (50°F–104°F)
Environmental humidity for operation	15%–93%RH
Environmental temperature for storage and transport	-25°C–55°C (-13°F–131°F)
Environmental humidity for storage and transport	≤ 93% RH
Environmental pressure	80KPa–106KPa
Battery life	Approx. 2 months at 3 tests per day

① **Note:** These specifications are subject to change without notice.

10. PACKAGE CONTENTS

- Upper arm blood pressure monitor.
- M-L size's fan shape cuff (upper arm circumference 22–42 cm)
C-ML-2242-FS-2.
- AAA batteries – 4 pcs.
- Mains USB adapter (depending on the set),
article AD-51/AD-81/AD-75.
- USB-C power supply cord.
- Storage bag (depending on the set).
- Gift box.
- Instruction manual.

11. SYMBOL INFORMATION

	FOLLOW INSTRUCTION BEFORE USE
	WARNING
	INFORMATION
	MANUFACTURER'S NAME
	ARTICLE NUMBER
	SERIAL NUMBER
	CE MARK WITH NUMBER OF NOTIFIED BODY
	BF TYPE DEVICE
	DISPOSAL FOR SEPARATE COLLECTION
	RECYCLABLE
	HOUSING INGRESS: WATER INGRESS PROTECTION RATE 21

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT
10°C — 40°C  OPERATION CONDITION	OPERATING CONDITION, TEMPERATURE
-25°C — 55°C  STORAGE CONDITION	STORAGE CONDITION, TEMPERATURE
15% — 93% 	STORAGE CONDITION, HUMIDITY
80 kPa — 106 kPa 	OPERATING CONDITION, ATMOSPHERIC PRESSURE
	MEDICAL DEVICE
	UNIQUE DEVICE IDENTIFICATION
	EC REPRESENTATIVE

12. WARRANTY INFORMATION

Warranty period is 2 years from the date of purchase for monitor and 1 year for cuff and adapter. This warranty doesn't cover any damages caused by improper using, and also battery and packaging. When a manufacturing defect is revealed during the warranty period a faulty unit would be repaired or, if repairing is impossible, replaced with another one. The warranty does not cover components and consumables subject to wear and batteries, bags, and package of the item. Manufacturing date is in a serial number: WWYXXXXX. WW – week, YY – year of production. XXXXX is the consecutive number in the batch. The manufacturer may change units partially or completely if necessary, without prior notice.

13. ELECTROMAGNETIC COMPATIBILITY INFORMATION

⚠ Use of the unit in conjunction with non-approved accessories can affect the device negatively and alter the electromagnetic compatibility.

- The essential performance:
 1. Limits of the error of the cuff pressure indication.
 2. Reproducibility of the blood pressure DETERMINATION.
 3. Alarm.

When electromagnetic interference affects the above performance, please stop using the device.

- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- It is suggested that the blood pressure monitor be kept at least 30 cm away from other wireless devices, such as WLAN unit, microwave oven, etc. It can't be used near active HF SURGICAL EQUIPMENT and the RF shielded room of an ME SYSTEM for magnetic resonance imaging, where the intensity of EM DISTURBANCES is high.

14. ADDITIONAL NOTES

- ① When measuring with the cuff, if the tester feels seriously uncomfortable, press the button of the blood pressure monitor to deflate the cuff, or remove the cuff directly from the arm.
- ① Do not stand while testing. Sit in a relaxed position while keeping your arm level with your heart.
- ① Avoid speaking or moving body parts while testing.
- ① When cleaning the unit, use a soft fabric and lightly wipe with mild detergent. Use a damp cloth to remove dirt and excess detergent.
- ① It is recommended the performance should be checked every 2 years.
- ① The equipment can be cleaned by lay operator according to the cleaning procedures in the instructions.
- ① The monitor, the batteries and the cuff, must be disposed of according to local regulations at the end of their usage.

Last revision 2025-W08

 **B. Well Swiss AG,**
Bahnhofstrasse 24, 9443 Widnau, Switzerland
www.bwell-swiss.ch

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**B.Well Swiss AG,**